



Quick Reference – Page 1

Automatic Braille Advance

Space with dots 1,2,4,5,6 (er)

Faster – Next Thumb Key

Slower – Previous Thumb Key

Battery Power Level – space with O, P

Block Command – space with B

Top of block – T

Copy block – C

Paste block – P

Move block – M

Delete block – D

Braille on / off – space with Next Thumb Key

Cursor Movement

Top of document – space with dots 1,2,3

Bottom of document – space with dots 4,5,6

Next character – space with dot 6

Previous character – space with dot 3

(read current character – space with dots 3,6)

Next word – space with dot 5

Previous word – space with dot 2

(read current word – space with dots 2,5)

Next sentence – space with dot 4

Previous sentence - space

Quick Reference – Page 2

Delete

Character – backspace with dots 3,6

Word – backspace with dots 2,5

Sentence – backspace with dots 1,4

Find Command – space with F

Find next occurrence – space with N

Force a new braille page – space with P

Force a new ink print page:

Space with 2,3,4,6 (the)

|

|

Space with P

Keyboard echo – space with O, K,

f for off

s for spell

w for word

b for both

Listen to entire file (go command) space with G

Line spacing – space with 2,3,4,6 (the)

|

L

Spacebar to “line spacing currently...”

D for double, or S for single



Quick Reference – Page 3

Main Menu – space with dots 1,2,3,4,5,6

Place Markers:

Define – backspace with D

Jump to – space with J

Delete – jump to, then press backspace

Speech on / off – space with Previous Thumb Key

Spelling Checker – space with dots 1,6 (ch)

Ignore all – I

Skip once – space with dot 5

Add to dictionary – A

Correct – C

Suggestion list – S

Stop Speech – enter with backspace

Switch between files – space with dots 1,2,5,6 (ou)

Next file in history list – space with dots 5,6

Previous file in history list – space with dots 2,3

Voice Quality:

Volume – enter with dot 1 or dot 4

Rate – enter with dot 3 or dot 6

Pitch – enter with dot 2 or dot 5